









 Kent Surgical Industries L.L.C Plot No.12, 54th Street, Companies Camp, Al Ain, Abu Dhabi, United Arab Emirates	This IFU Cover The following products: Endoscopy Channel Pre-Cleaning Detergent with Sponge (ES02).	Ph.: +971 3 753 2555 Email: support@kentsurgical.com Website: www.kentsurgical.com
--	---	--

Symbols Using in Labeling:

Symbol	Symbol Title	Description of symbol
	Catalogue number	Indicates the manufacturer's catalogue number.
	Consult instructions for use.	Indicates the need for the user to consult the instructions for use.
	Date of manufacture	Indicates the date when the medical device was manufactured.
	Do not use if package is damaged	Indicates a medical device that should not be used if the package has been damaged or opened.
	Medical device	Indicates the item is a medical device
	Manufacturer	Indicates the medical device manufacturer.
	Batch code	Indicates the manufacturer's batch code.
	Do not re-use	Indicates a <i>medical device</i> that is intended for one <i>single use</i> only.
	Use-by date	Indicates the date after which the medical device is not to be used.
	Temperature limit	Indicates the temperature limits to which the medical device can be safely exposed.

Intend of use for Endoscopy Channel Pre-Cleaning Detergent with Sponge of Kent Surgical®:

- A Cleaner for the initial flush and wipe of an endoscope.

Product Specification:

- Clear, Blue liquid. Non-flammable, with medical-grade foam Sponge.

General Safety Information:

- Read and follow these instructions and keep them accessible to the staff working with the product.
- The product should be used by trained and qualified personnel.
- The products must be used exclusively for the intended purpose by appropriately trained and qualified personnel.
- Wash thoroughly after handling.

Warning:

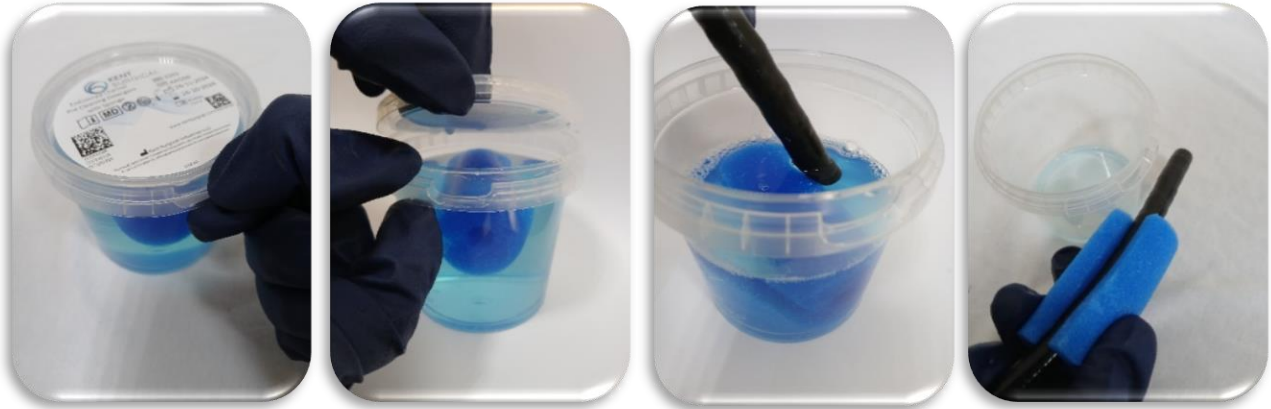
- Do not breathe vapor, mist, or spray.
- Use your Personal Protective Equipment (PPE) during using the product.
- If in Eyes; Rinse cautiously with water for several minutes. Remove contact lenses, if present, and easy to do. Continue rinsing.
- Do not use it if the package is damaged.
- Do not use the expired product.
- Store it between 5°C & 25°C.
- For more safety instructions check the safety data sheet.

Instructions For Use:

All scopes must be pre-cleaned with detergent, as soon as the endoscope has been removed from the patient. Before the endoscope is even detached from the light source or processor.

Each endoscope manufacturer will have slightly different guidelines on how to flush their scopes, but all are clear that this has to be carried out immediately after the procedure.

1. Once the procedure is finished, Break the lid lock.
2. Open the Endoscopy Channel Pre-Cleaning Detergent pack.
3. Suction up the Endoscopy Channel Pre-Cleaning Detergent until the pack is empty.
4. Remove the scope sponge and wipe down the insertion tube.

**STEP ONE**

Break the lid lock.

STEP TWO

Open the easy-peel lid.

STEP THREE

Suction up the detergent until the pack is empty.

STEP FOUR

Remove the scope sponge & wipe down the insertion tube.

Disposal:**Waste treatment methods****Recommendation**

Must not be disposed of together with household garbage. Do not allow the product to reach the sewage system.

Uncleaned packaging:**Recommendation:**

Disposal must be made according to official regulations.

Recommended cleansing agents: Water, if necessary, together with cleansing agents.